# Team-8 Stretcher

**Contents:**

- x1 Team-8 Stretcher, complete with securing straps and drag harness
- x1 Extension Strap

NSN: 6530-99-359-4978

Does not contain or presence of natural rubber latex

Non-sterile

Consult instructions for use

Manufacturer

Safeguard Medical Technologies Ltd.  
The Old Rectory  
Hope Under Dinmore  
Herefordshire, HR6 0PW  
United Kingdom

Importer

Authorised Representative in the EC

Safeguard Technologies Ltd,  
Willow Grove,  
Delgany, A63 XY89  
Co Wicklow,  
Ireland

# Team-8 Stretcher

## Instructions For Use

### Intended use:

The Team-8 Stretcher is intended to be used to extract (or evacuate) casualties as quickly and safely as possible in an emergency or hazardous situation.

### User profile:

The Team-8 Stretcher is designed for use by any person in an emergency situation where rapid extrication is required.

### Patient profile:

The Team-8 Stretcher is used to take patients who need assistance out of a hazardous situation in as quickly a time frame as possible.

### Clinical benefits:

- Rapid extrication from hostile or restrictive area
- Physical support to help immobilise the patient
- Security – reduce likelihood of patient being dropped and further injured
- Controlled movement over hazardous terrain

### Product claims:

- Lightweight
- Compact
- Quick casualty evacuation
- Safe
- Lifiable
- Robust

### Contraindications:


The Team-8 Stretcher is not intended to be used for spinal immobilisation. There is risk of damage to the spinal cord potentially leading to irreversible neurological damage.

The Team-8 Stretcher is not intended for high angle rescue.

### Residual risks/warnings:


- Before use the Team-8 Stretcher should be checked to ensure material it is within the shelf-life and material deterioration has not occurred or that the device is contaminated with bodily fluids. Failure to do so can risk damage to the patient and patient or user cross-contamination

- For use up to 24 hours or until definitive care is reached. Prolonged time on-stretcher time may result in pressure sores. Consideration should be made to local procedures and guidelines regarding care of skin pressure areas
- Do not use solvents, neat bleach, phenolic based cleaning solutions or abrasive products to clean as this may cause material deterioration or user/patient skin reactions
- Before any re-use or disposal, proper cleaning should be completed in line with the cleaning methods in this IFU. Failure to do so can risk patient or user cross-contamination
- Any serious incident that has occurred in relation to this device should be reported to manufacturer ([vigilance@safeguardmedical.com](mailto:vigilance@safeguardmedical.com)) and applicable national competent authority.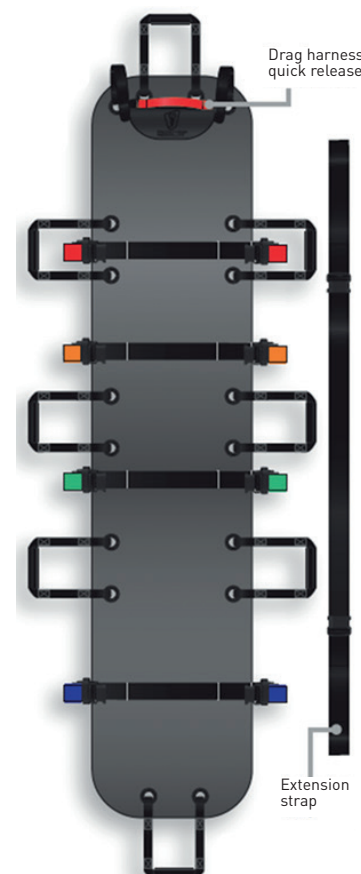

### INSTRUCTIONS:

1. Before using the Team-8 Stretcher it should be checked to ensure material deterioration has not occurred from previous use and that any biological fluids from previous persons have been removed. Do not use if overly damaged.
2. Remove the outer extension strap and position the unrolled stretcher alongside the casualty (shiny side down) with the casualty securing straps visible on the uppermost surface and the drag harness pouch at the head end, as in Figure (A).
3. Either slide the stretcher underneath the casualty or move the casualty onto the stretcher.
4. Pull the coloured tabs to reveal the securing straps.
5. Fasten the top securing strap (red tabs) across the chest with the arms outside the straps. Depending on operational procedures, this may be sufficient in a hostile environment to begin to move the casualty using the drag harness found in the drag harness pouch (B). Secure the remaining colour paired straps, and tighten as required per Figure (C).
6. The extension strap has compatible material and buckles so can be used to lengthen any of the casualty securing straps, e.g. the chest strap for an exceptionally large casualty wearing body armour, or the foot strap to create a figure-of-eight strapping system for added security.
7. Once patient is secure for extraction, you can lift the patient using the carry handles or drag using the drag harness. The Team-8 can be used to carry a casualty on a rigid stretcher such as a Scoop stretcher by passing the securing straps through the holes on the scoop (D).

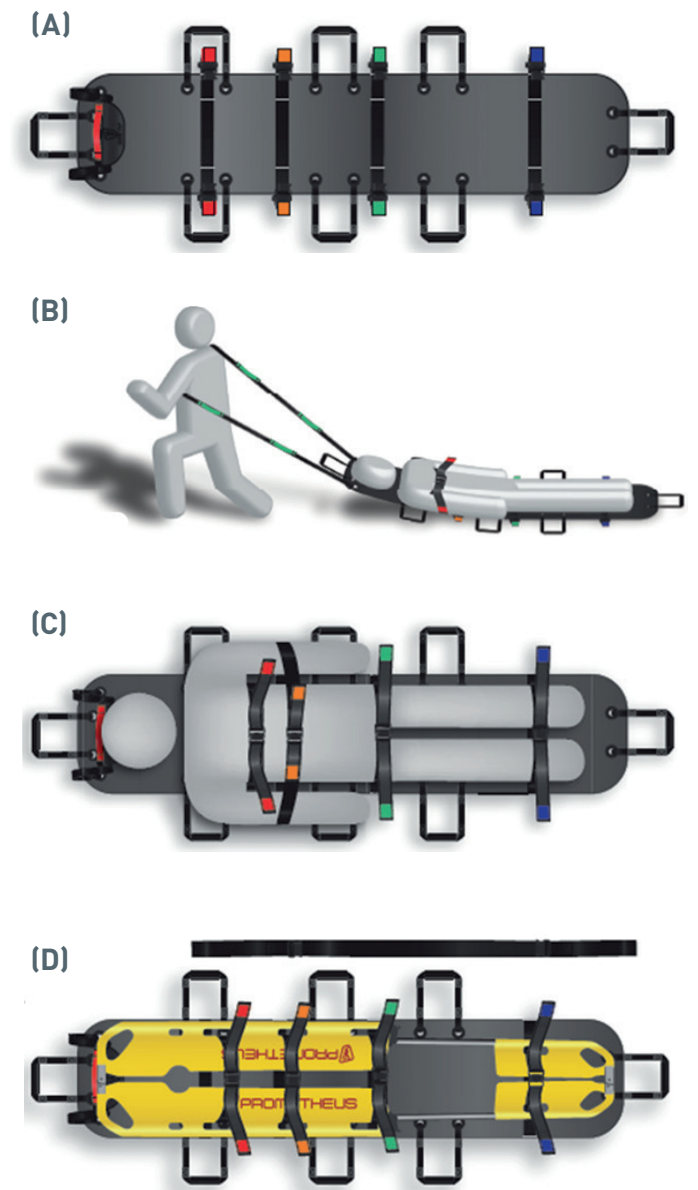### CLEANING METHODS:

Cleaning: Before any re-use or disposal, proper cleaning should be completed in line with the following cleaning method (NHS Cleaning Methods available on request).

1. Wipe all surfaces down with a mild detergent diluted in warm water (40°C).
2. If equipment has come into contact with blood or bodily fluids, wipe surfaces down with 0.1% Chlorine solution and subsequently wipe down with clean cloth with water after 5 minutes.
3. Care should be taken to ensure material deterioration has not occurred.

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